

Restoring a body.  Restoring a life.



Shape an
ideal path to
reconstruction.

MENTOR® CPX™ 4
Breast Tissue
Expander

mentormedical.co.uk
yourbreastoptions.co.uk

 **MENTOR®**
Make life more beautiful™



CHOOSE THE TISSUE EXPANDER THAT CONSIDERS YOUR NEEDS.

Unique design features and enhancements optimize performance for expansion results you and your patients will welcome.

MENTOR® CPX™4 BREAST TISSUE EXPANDERS

EXPERIENCE THE DIFFERENCE THROUGHOUT THE EXPANSION CONTINUUM.

INTRAOPERATIVE

*Smooth, flat injection dome and enhanced shell pliability**

- Helps ease insertion and removal through incision

Optional suture tabs at 3, 6 and 9 o'clock positions

- Stabilize placement to help control aesthetic outcome
- Allow left to right alignment in bilateral procedures



DURING THE EXPANSION

BufferZone™ Self-Sealing Patch and posterior Dacron® Patch

- Focus expansion towards the lower pole of the breast

*More pliant proprietary BufferZone™ Patch, smooth injection dome surface and a stronger magnet**

- Design for enhanced patient comfort
- Prevent leakage in the event of an accidental needle puncture

RELIABLE 2-STAGE RECONSTRUCTION

BufferZone™ and Dacron® Patch

- Drive lower pole expansion to mimic the shape of the natural breast

SILTEX® Texture

- Prevent pooling or sequestration of body fluids
- Provide disruptive surface for collagen interface

Implant Matching

- Matching range of definitive implants to be used after tissue expansion

*Compared with the MENTOR® CPX2™ and CPX3™ Breast Tissue Expanders.

MENTOR® CPX™4 Breast Tissue Expanders

Optionally available with suture tabs at 3, 6 and 9 o'clock positions

FOR MORE INFORMATION OR
TO PLACE AN ORDER,
visit mentormedical.co.uk

THREE CONTOURED SHAPES ACCOMMODATE ALL YOUR NEEDS.



MENTOR® CPX™4 Breast Tissue Expander Style 8100 - Low Height					
CATALOG	VOLUME	WIDTH	HEIGHT	PROJECTION	
3548111	250	11.4	8.1	6.1	
3548112	350	12.7	9.4	6.5	
3548113	450	14.0	10.2	7.1	
3548114	550	15.0	10.9	7.4	
3548115	650	15.7	11.2	7.9	
3548116	750	16.5	11.9	8.1	



MENTOR® CPX™4 with Suture Tabs Breast Tissue Expander Style 9100 - Low Height					
CATALOG	VOLUME	WIDTH	HEIGHT	PROJECTION	
3549111	250	11.4	8.1	6.1	
3549112	350	12.7	9.4	6.5	
3549113	450	14.0	10.2	7.1	
3549114	550	15.0	10.9	7.4	
3549115	650	15.7	11.2	7.9	
3549116	750	16.5	11.9	8.1	



MENTOR® CPX™4 Breast Tissue Expander Style 8200 - Medium Height					
CATALOG	VOLUME	WIDTH	HEIGHT	PROJECTION	
3548211	275	10.7	9.3	6.2	
3548212	350	11.7	10.0	6.6	
3548213	450	12.7	10.8	7.0	
3548214	550	13.5	11.7	7.4	
3548215	650	14.6	12.6	7.6	
3548216	800	15.6	13.3	8.0	



MENTOR® CPX™4 with Suture Tabs Breast Tissue Expander Style 9200 - Medium Height					
CATALOG	VOLUME	WIDTH	HEIGHT	PROJECTION	
3549211	275	10.7	9.3	6.2	
3549212	350	11.7	10.0	6.6	
3549213	450	12.7	10.8	7.0	
3549214	550	13.5	11.7	7.4	
3549215	650	14.6	12.6	7.6	
3549216	800	15.6	13.3	8.0	



MENTOR® CPX™4 Breast Tissue Expander Style 8300 - Tall Height					
CATALOG	VOLUME	WIDTH	HEIGHT	PROJECTION	
3548311	250	10.1	10.7	5.6	
3548312	350	11.3	11.8	6.0	
3548313	450	12.3	12.9	6.5	
3548314	550	13.2	13.8	6.9	
3548315	650	14.0	14.6	7.3	
3548316	750	14.6	15.3	7.6	
3548317	850	15.4	15.9	7.9	



MENTOR® CPX™4 with Suture Tabs Breast Tissue Expander Style 9300 - Tall Height					
CATALOG	VOLUME	WIDTH	HEIGHT	PROJECTION	
3549311	250	10.1	10.7	5.6	
3549312	350	11.3	11.8	6.0	
3549313	450	12.3	12.9	6.5	
3549314	550	13.2	13.8	6.9	
3549315	650	14.0	14.6	7.3	
3549316	750	14.6	15.3	7.6	
3549317	850	15.4	15.9	7.9	

Individual implant dimensions may vary slightly in products of this type. Not all units conform exactly to the dimensions noted above.

Important Safety Information:

MENTOR® Breast Tissue Expanders are used for breast reconstruction following mastectomy. These expanders are intended for temporary subcutaneous or submuscular implantation and are not intended for use beyond six months. The MENTOR® Breast Tissue Expanders are devices that contain magnetic injection domes and are NOT MRI compatible. Do not use the MENTOR® Breast Tissue Expander in patients where an MRI may be needed. DO NOT use the MENTOR® Breast Tissue Expander in patients that have a previously implanted device that could be affected by a magnetic field. For detailed indications, contraindications, warning and precautions associated with the use of all Mentor implantable devices, which include MENTOR® Saline Filled Breast Implants, MENTOR® CPG™ Shaped Breast Implants and MENTOR® Breast Tissue Expanders, please refer to the Product Insert Data Sheet provided with each product, or review the Important Safety Information provided at www.mentorwllc.eu.



Corporate Headquarters

Santa Barbara, CA 93111 USA
Tel: +1 805 879 6000
www.mentorwllc.com

United Kingdom & Ireland
Johnson & Johnson Medical Ltd
Tel: +44 (0)800 973 082
Fax: +44 (0)134 486 4122
mentormedical.co.uk