



BRCDC

Sterile Breast Reduction Compression Device



The Breast Reduction Compression Device (BRCDC) is a smart product to facilitate the breast reduction surgeries. The product consists of an adjustable nylon strap encased in silicone tubing. The silicone tubing prevents the strap from slipping out of place. Also, the silicone tubing minimizes the stress on the tissue.

All BRCDC devices are delivered sterile in a double package.

All BRCDC devices are CE marked and for single use only.



Procedure

- Wrap the BRCDC around the base of the breast and put the band into a loop.
- Tighten the breast by contracting the band.
- Release the BRCDC by pulling back the small plastic tab at the head.

Sizes

• BRCDC Standard
Procedure: 500-800 g reduction
Strap dimensions: 1.5 x 50 cm

• BRCDC XL
Procedure: Up to 1300 g reduction
Strap dimensions: 1.5 x 70 cm

